

AIXIN MEDICAL

USER MANUAL

*AX-MTB Model Milling
Machine*


Description:

This machine is a necessary equipment for trimming the sides of plaster models. It consists of a case, a motor, a durable alloy grinding head, a power switch and an indicator light.

Specifications:

Power supply voltage	AC 220V / 110V 50HZ / 60HZ±10%
Power	140W
Grinding head	C1625 M06
Dimensions	26×21×25 (cm)

Diagram:

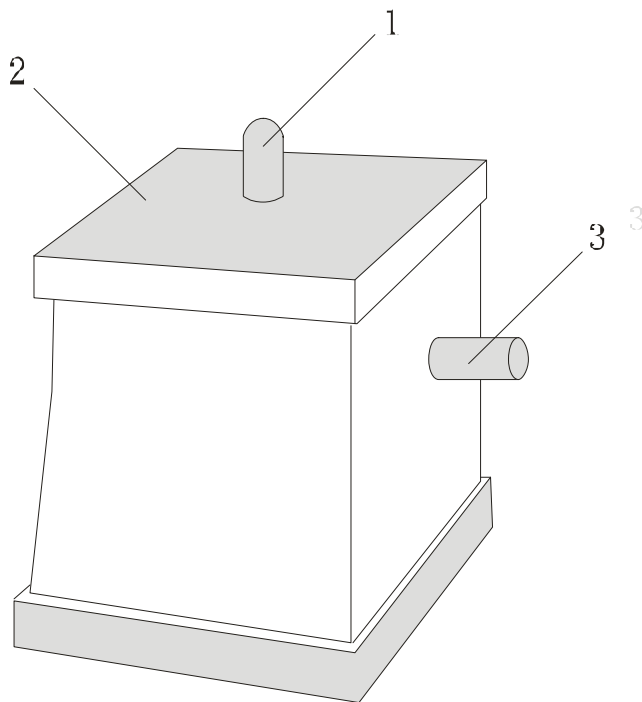

1. Grinding head
2. Working platform
3. Dust outlet

Instructions:

1. Unpack the packages, check if all the accessories are in the package. Read this manual carefully before using.
2. The electric outlet must have a good grounding.
3. After connecting to the power supply, the indicator light on the lower left corner of the machine should be on.

4. Turn on the power switch, the motor (grinding head) should start working and the red indicator light in the power switch is on.
5. While grinding, push with both hands with gentle and evenly, do not touch the grinding head.
6. Users can connect the dust outlet with a pipe into a dust box, connecting with a suction box is preferable.
7. Clean the dust in the dust chamber regularly.

Cautions:

1. If the motor does not work, there might be two possibilities.
 - a. If the green indicator light is not on, check if the power supply is connected and whether the fuse tube in the electric outlet is burned out.
 - b. If the green indicator light and the power indicator are on, check if there is open circuit in the motor, capacitor and circuit.
2. If the shaking is severe, it could be caused by the loosening of the nut on the clip. To resolve the problem, lift the cover and tighten the loose nut.