



MLG TCF

INTRAORAL CAMERA CF-680


INSTRUCTIONS

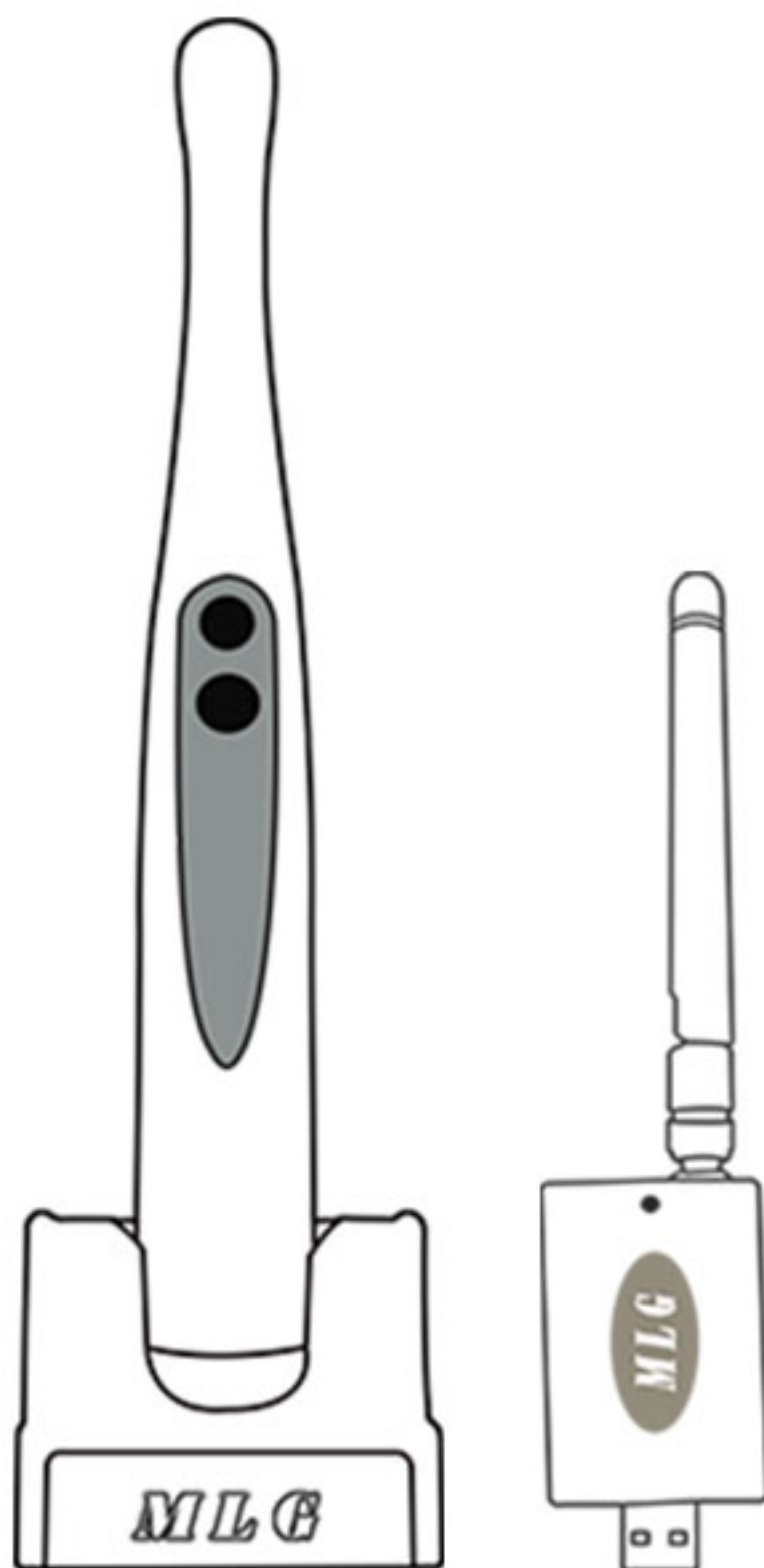

KEY FUNCTION

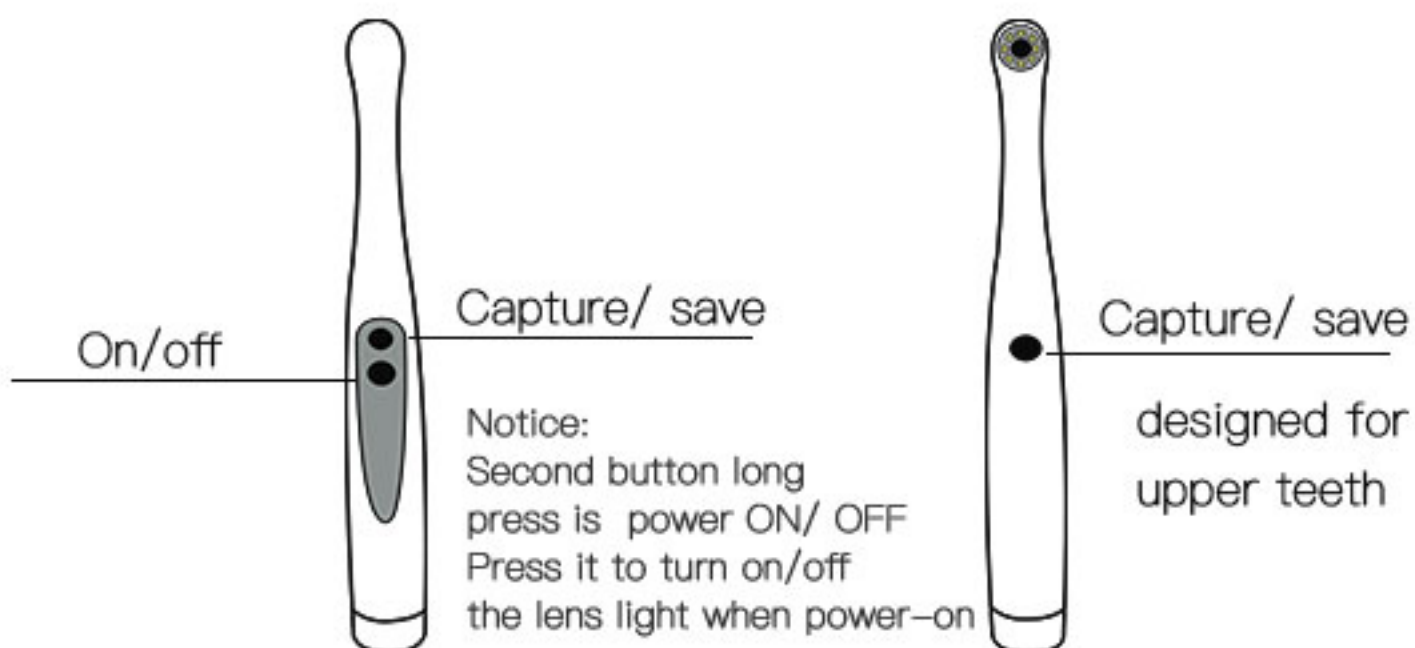

STEP

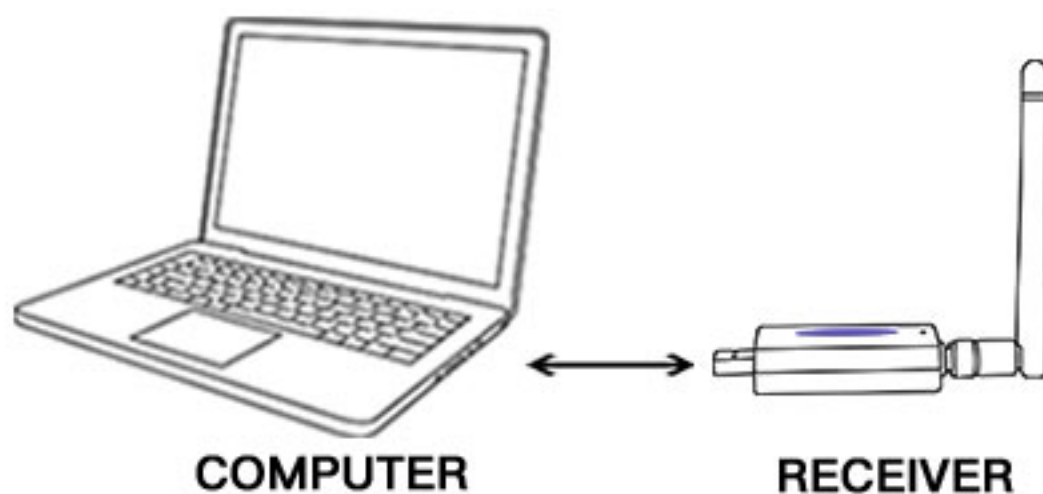

1. Receiver connect to the computer
2. press the power on button 2 seconds and the green light will lights up ,Blinking is already running
3. make a settings to connect WIFI named: first use password is "88888888"
4. Open the management software to take pictures

PARAMETER

Name: Intraoral camera

Model: CF-680

Pixel: 13.0 Mega

Output: USB 2.0

Voltage: 5V

light : 8 pcs LED

Focus range : 3-50MM

Angle of view: 105°


PACKING LIST



Handpiece



camera hold base

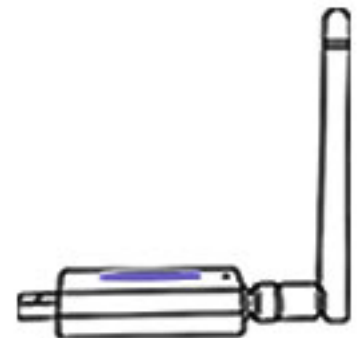

Receiver



USB cable




Sleeves




manual

PRODUCT PROFILE


Handpiece

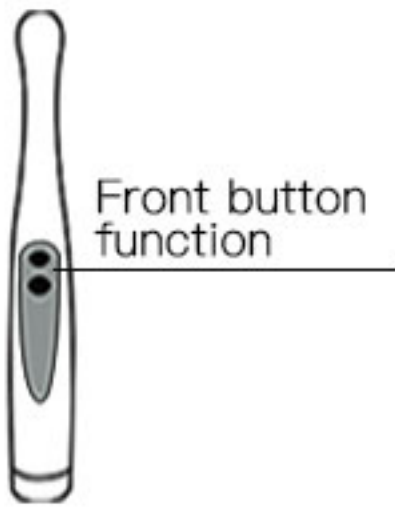

Bottom of handpiece

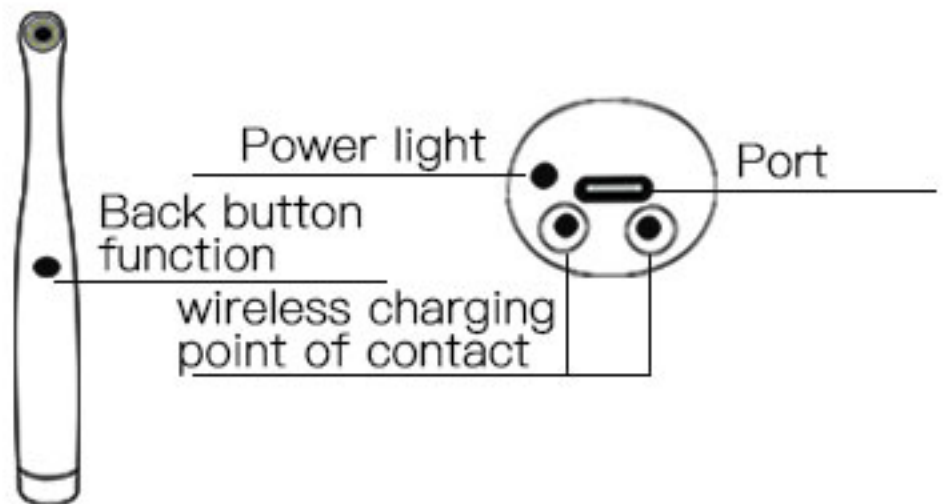


front of base

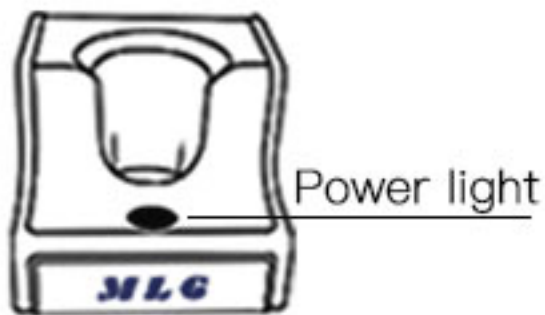

Back of base



receiver

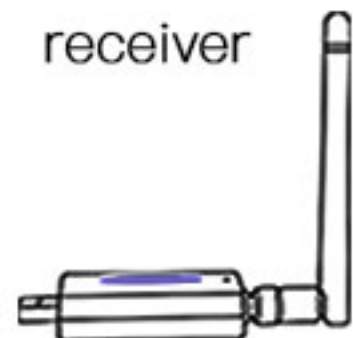

NOTICE

Please read the following precautions carefully before using this product

Product

- 1.Do not touch the signal connector with sharp objects, metal or liquid to avoid short circuit and product damage.
- 2.If you have not used this product for a period of time, please make it power off please do not disassemble, which may damage the product.

Power

- 1.Please use the original manufacturer specified special power transformer, so as not to damage the product.
- 2.If you find the power cord or outlet is damaged, please do not use it again.

Use environment

- 1.Please put the product in a dry and ventilated place.
- 2.Please do not put this product on a place are too cold, too hot or too humid, direct sunlight, dust and more places.

HOW TO SOLVE TROUBLES

If there are some troubles of products, Please see below special case firstly. If still can't be solved, please contact your local dealer.

the products can't be start-up when open (ON/OFF)switch)	1.check the power socket if it is ok 2.check the cable if it is connected
computer or TV can't reveal images	1.check the connection between products and computer or TV if it is correct 2.check the Driver Setup if it is finished,
in the process of use, the image will be a little blurry	if you use it in the first time, it belongs to natural reaction. It will be better later on
the images of computer or TV have spots	1.check the screen if it has spot 2.check the lens if it has spot 3.if the spot can't be cleaned, please contact the local dealer

WARRANTY

Thank you for purchasing . In order to ensure our company's high-quality after-sales service, the following are instructions for warranty.

Product Warranty Conditions

- 1.The warranty period of the product is 1 year. During this period, if any problem of product, can inquiry the dealer or our company for consultation and maintenance.
- 2.The following cases are excluded from the warranty service , and we will charge a fee (charge standard = maintenance fee + replacement part fee + transportation fee or postage fee)
 - 1)The quality problems because of improper use or damage to follow product instructions
 - 2)When the product damage, replacement by consumers, dismantle it or any repair station without the company's approval.
 - 3)artificial damage, or damage caused by installation, movement, etc.
- 3.This product has passed the relevant quality management inspection as a qualified product before leaving the factory, but factors such as transportation cannot be ruled out. Once product breaks down, you can contact the dealer or our company with the warranty sheet to the warranty service
- 4.customer service hotline 0788-33933718/33813513