



## JT-19C Trimmer Operation Manual

This powerful trimmer comes with low noise .Power: 500W , Rotation Speed : 2800rpm , Power Supply:220V/50HZ ( 110V/60HZ )。 The overall structure of the motor, install and support has obtained national patent for utility model.

Working Style: ① Grinding with water。 ②Quartz sand/ Diamond disc, The standard configuration: Diamond disc ③ Abrasive disc diameter of bore:  $\phi 25.5/\phi 32$

Place the machine in a plain place , connect it with water supply 。 ( if grinding without water , no need to connect water supply , only need connect the dust-collector pipe with dust-collector )

- 1、 Power on, Four hole on either side of the base to support customer installation aspect screws), connected to the inlet pipe (work pressure pipe controlled by electromagnetic valve, and synchronous motor switch), and out of the pipe
- 2、 It can work at three different angle , you can adjust it by adjusting screw(7)。
- 3、 How to change the disc 。 Power off, Open the front cover(11), disassemble the disc(1) ( If not convenient enough ,you can also disassemble the water spray tube(13) ) you can change the disc easily in this way。
- 4、 When duty off, clear up the dust timely pls。

### Trouble Killing

- 1、 If there is some abnormal noise when working on

Possible reason: ① Axon is loosing ② Disc is not plain enough。

How to solve it: ①Dissemble the disc(16) with spanner, make the snap Ring tightly; ② change the disc。

- 2、 The machine does not work when power on

Possible reason: ① fuse get burnt; ② motor get burnt; ③ switch get wrong。

How to solve it: ① change the fuse (22) which inside the socket; ② change the motor; ③ change the switch。