



JT-51 Specification of Powerful Dental Vibrator



This is a big power ,high efficiency vibrator, the unique design of heat radiation base make it always work in long time . It has two grades, weak grade, strong grade, can be changed as necessary.

Before power on, put the switch in the middle part "O",and then power on, turn on the switch(the indicator light comes on), and choose the strong grade or weak grade,finally turn on the switch two or switch one, it starts to work.

Breakdown phenomenon: when it works, the noise becomes large suddenly, comes with unusual sound meantime.

Reason : the gap between vibrate's platform and transformer is unusual .

Trouble shooting: Cut down the power, remove the rubber platform, and reverse the machine, put the base up, use "+" type screw-driver to remove 4 pieces M6 screws, in order to make vibrator's platform separating from the body. .Inspects four vibrate support pole one by one whether they have become loosen or the height are not same. Readjust the four vibrate support poles' height to be 68mm,, and lock them tightly. (Adjust tool: 10mm opposite side hexagonal socket spanner)



JT-51B

Powerful Dental Vibrator

序	零件号	数量
1	rubber plat	1
2	platform	1
3	transformer	1
4	vibrate support pole assembly	4
5	two in on plug	1
6	board	1
7	main body	1
8	rubber foot support	4
9	speed regulation potentiometer	1
10	three grades switch	1
11	knob	2
12	switch	4

