


Product use: This product uses violet light to kill microorganisms. It is suitable for eliminating microorganisms on glass, metal and plastic tools; it provides a clean and hygienic environment for storing tools and so on.

Product switch description: as shown below

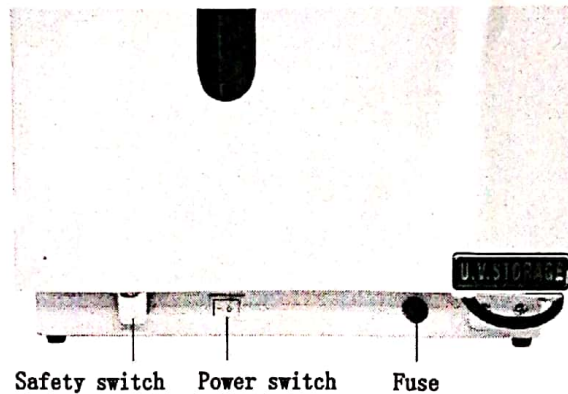

Instructions:

1. Place the tool on the tray evenly after the tool is cleaned and dried.
2. Place the machine on a stable place or hang it vertically on the wall.
3. Be sure to close the door. Before turning on the power, make sure that the door control is turned on.
4. Check whether the lamp is working. If it does not work, open the door after turning off the power and tighten the lamp to ensure that it is well connected with the socket.
5. The cabinet is equipped with a door control system, and the power is automatically cut off when the door is opened.
6. Working for 15 minutes is enough to kill the microorganisms on the tool.

Tips:


1. Not suitable for destroying liquid microorganisms.
2. The product has a door control system. For safety, please cut off the power before opening the door.
3. When cleaning the cabinet, please use a wet towel, please do not wash it directly with water.
4. If there is any failure during use, please contact the maintenance staff at any time. Non-professionals should not disassemble.

Product parameters:

Model	XS-052
Power	15W
Voltage	220V
Frequency	50Hz
Volume	27L
Working time	15min
Size	350X260X550mm


Application: This product uses UV light to kill microorganisms. Sterilize for microorganisms of the glasses, metal and plastic.

Description: as below

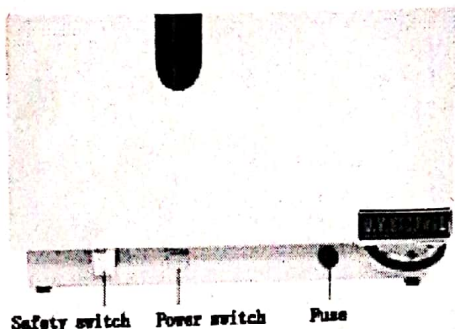

Product parameters:

model	XS-052-1
Power	15W
Voltage	220V
Frequency	50Hz
capacity	27L
Working time	15min
Size	350X260X550mm

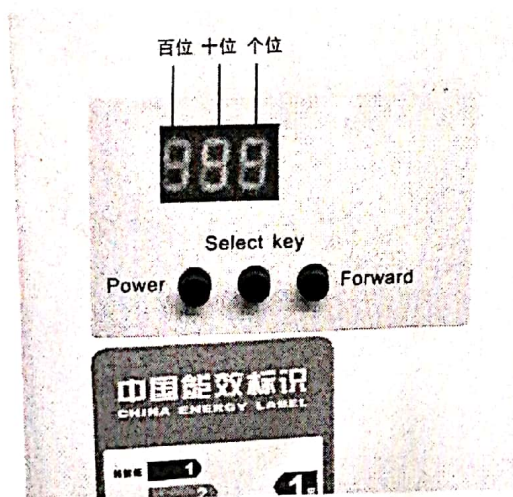

- 1、The number on the display screen is working time (minute)
- 2、Can setting for 0-999 minutes.
- 3、Turn on the power then open the switch. Press the power can be working, second press will be stop working.
- 4、Select key for time adjust, press the first key and press the Forward key can be adjust the time. The second and the third key as like the first one. When you setting well how many time you want to sterilizer the display will be twinkling, then it's ready to sterilizer.
- 5、Press the Power key, the unit can be working.

Instructions:

1. Place the tool on the tray evenly after the tool is cleaned and dried.
2. Place the machine on a stable place or hang it vertically on the wall.
3. Be sure to close the door. Before turning on the power, make sure that the door control is turned on.
4. Check whether the lamp is working. If it does not work, open the door after turning off the power and tighten the lamp to ensure that it is well connected with the socket.
5. The cabinet is equipped with a door control system, and the power is automatically cut off when the door is opened.
6. Working for 15 minutes is enough to kill the microorganisms on the tool.

Tips:

1. Not suitable for destroying liquid microorganisms.
2. The product has a door control system. For safety, please cut off the power before opening the door.
3. When cleaning the cabinet, please use a wet towel, please do not wash it directly with water.
4. If there is any failure during use, please contact the maintenance staff at any time. Non-professionals should not disassemble.