

JT-29B 蠟刀加熱器

Environment for Use 使用環境:

--indoor use

室內使用

--Temperature : Between 5°C and 40°C

溫度 5°C--40°C

--Moisture : Between 50%-- 80%

濕度: 50%~80 %

--Tolerance of °C ±10% of designated voltage

額定電壓±10%

--Pollution Grade:2

污染等級: 2

Specifications 規格:

CLASSIFICAT ON 分類	INPUT POWER SOURCE 電源	AMP 功率	FUSE 電流	TEMPERATU RE 溫度	DIMENSION 體積(MM)			WEIGHT 重量 (KG)
JT-29B -A	AC220-240V 50/60Hz	100W	2A	--	92	270	95	0.55
JT-29B -B	AC100-120V 50/60Hz	100W	2A	--	92	270	95	0.55

Description of Products 產品說明

It is a tool to apply high heat to the carving knife instantaneously in safe and convenient manner rather than the type that the carving knife is heated by the fire tools such as Bunsen burner, alcohol lamp, and gas burner etc at time of waxing up works

它是一個工具，適用於蠟刀瞬間加熱，安全、方便。區別於傳統煤氣燈、酒精燈、煤氣燃燒器等工具。

*High heat can be applied to the carving knife instantaneously

瞬間對蠟刀產生高熱量

*It is safe against burns and fires

沒有明火，安全可靠。

*Energy can be used efficiently due to the operation by a sensor

由於傳感器的作用，能量可以被有效地利用。

*It is ergonomically designed for the convenient use

它符合人體工程學設計，使用方便。

*It can be cleanly used as there is a replaceable cap

清潔設計、可更換橡膠套。

*In case carving knife is inserted for a long time, error sign is operated to protect the circuit

safe for the safety of the product and its user

蠟刀長時間放置，機器會啟動保護電路，保護產品和使用者的安全。

How to use 使用說明

1) Plug in the power cord and turn on the switch.

插上電源線、打開開關

2) Hold the wax knife and place the wax knife in the center position of the sensor. The handle is close to the optical sensor and it will be removed from the wax knife for 2-3 seconds.

手持蠟刀，將蠟刀放入感應器中心位置，手柄處接近光學傳感器，持續 2-3 秒鐘後移出蠟刀。

3) The wax knife gets hot, the machine works normally, and the operator can carry on the follow-up work.

如果蠟刀變熱，機器運轉正常，操作人員可以進行後續工作了。

4)If there is no heat in the wax cutter, contact the supplier to solve the problem.

如果反復感應蠟刀扔沒有熱量，請聯繫供應商解決問題。

Caution in usexc 注意事項:

1)Do not touch the product with wet hands.

不要用濕手觸摸產品。

2)Precise circuit is built in this product .Excessive power or shock to the product can cause a failure.

產品內置精密電路，對產品過渡的動力或衝擊會導致故障。

3)If there is a question or defect found during the use of a product do not directly disassemble it ,
and inquire to the place of purchase.

如果在產品使用過程中發現問題或缺陷，請不要直接拆卸，詢問購買地。

4)For better performance, avoid the sensor from interfering by infrared(ie. Solar or heat lamp etc)if possible
為了獲得更好的性能，避免傳感器有紅外干擾（如太陽能或加熱燈等）如果可能的話。

5)Be careful not to place the carving handle stained with melted wax teach the optical sensor.
(This can deteriorate the sensitivity of the optical sensor)

注意不要將蠟刀內熔化的蠟沾到光學傳感器上。（這會影響光學傳感器的靈敏度）

6)Do not use damaged power cord or plug, loose outlet (There is a danger of electric shock fire).

不要使用損壞的電源線或插頭、鬆動的電源插座，（有觸電和火災危險）。

7)Do not bend the bend the power cord excessively or be squashed by a heavy object
(There is a danger of electric shock ,fire).

不要過度彎曲電源線或被重物擠壓（有觸電和火災危險）。

8)Plug must be connected to the outlet that has an earth terminal
(There is danger of electric shock ,fire).

插頭必須連接到有接地端子的插座上（有觸電和火災危險）。

9） Do not block the ventilator located at the bottom of the product (It can overheat,fire)
不要阻塞衛浴產品底部的通風機（會過熱或起火）

10） Unplug the cord when it lightens and thunders, and when it is not used for a long time

(There is a danger of electric shock, fire)

雷雨天氣時，長時間不使用應該拔掉電源線。（有觸電和火災危險）

11） Do not pull out the cord area or touch the power plug with wet hands
(There is a danger of electric shock,fire)

不要拔出電線區域或用濕手觸摸電源插頭（有觸電和火災危險）

12） Do not install on the place where there is much Moisture or dust, and the place where
water splashes (There is a danger of electric shock, fire)

不要安裝在潮濕或灰塵較多的地方和水濺的地方。（有觸電和火災危險）

13） Do not install on the place near the electric heating appliance . (There is a danger of fire)
不要安裝在電熱器具附近的地方。（有火災危險）

14） Unplug the power cord ,and do not directly spray water or wipe it with chemicals etc
(There is a danger of electric shock, fire)

拔掉電源線，不要直接噴水或用化學藥劑擦拭（有觸電和火災危險）。

15） Wipe the dust, water ect.Well off the power plug pin and contact area
(There is danger of electric shock, fire)

擦去插頭和接觸區的灰塵和水（有觸電和火災危險）。

16） If there is a strange sound, smell, smoke found from the product, unplug the
power cord immediately (There is danger of electric shock, fire)
如果產品發出奇怪的聲音、氣味、煙霧。應立即拔掉電源線。（有觸電和火災危險）。

17） Do not disassemble, repair , remodel, carelessly (There is danger of electric shock)
不要隨意拆卸、修理、改造產品。（有觸電和火災危險）

*Manufacturer will never be responsible for any problems occurred by the misuse
of the product in a manner other than prescribed ones

*生產商對使用者不按規定使用產品，所產生的問題，不負任何責任。