



TR-966 Dental Operating LED Light Manual

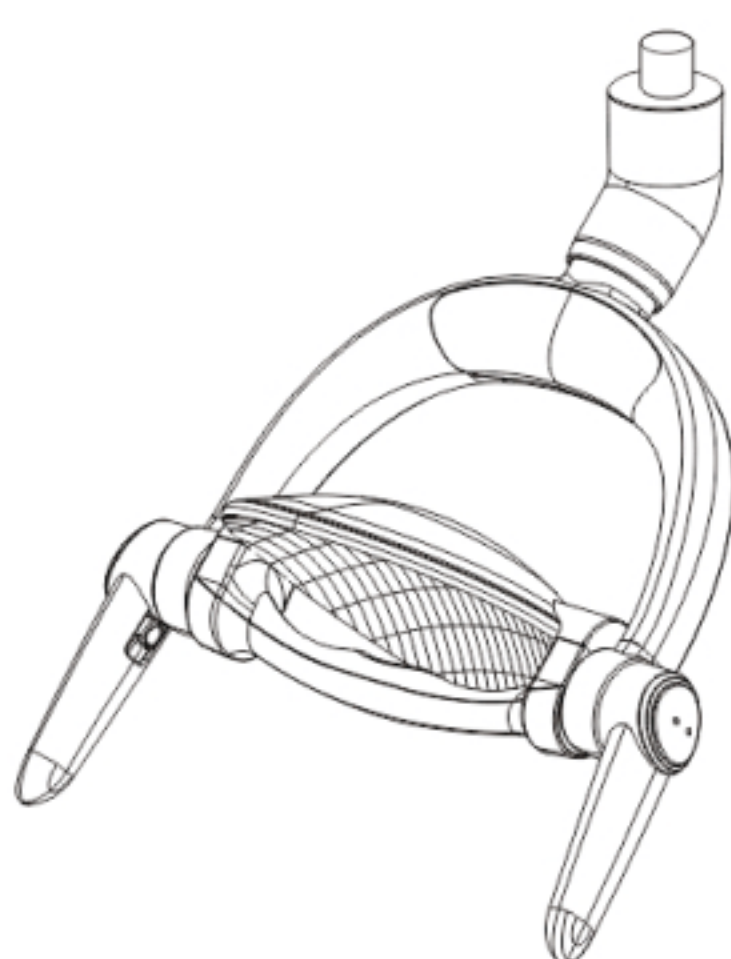

Please read the manual carefully before the installation.

I. The structure

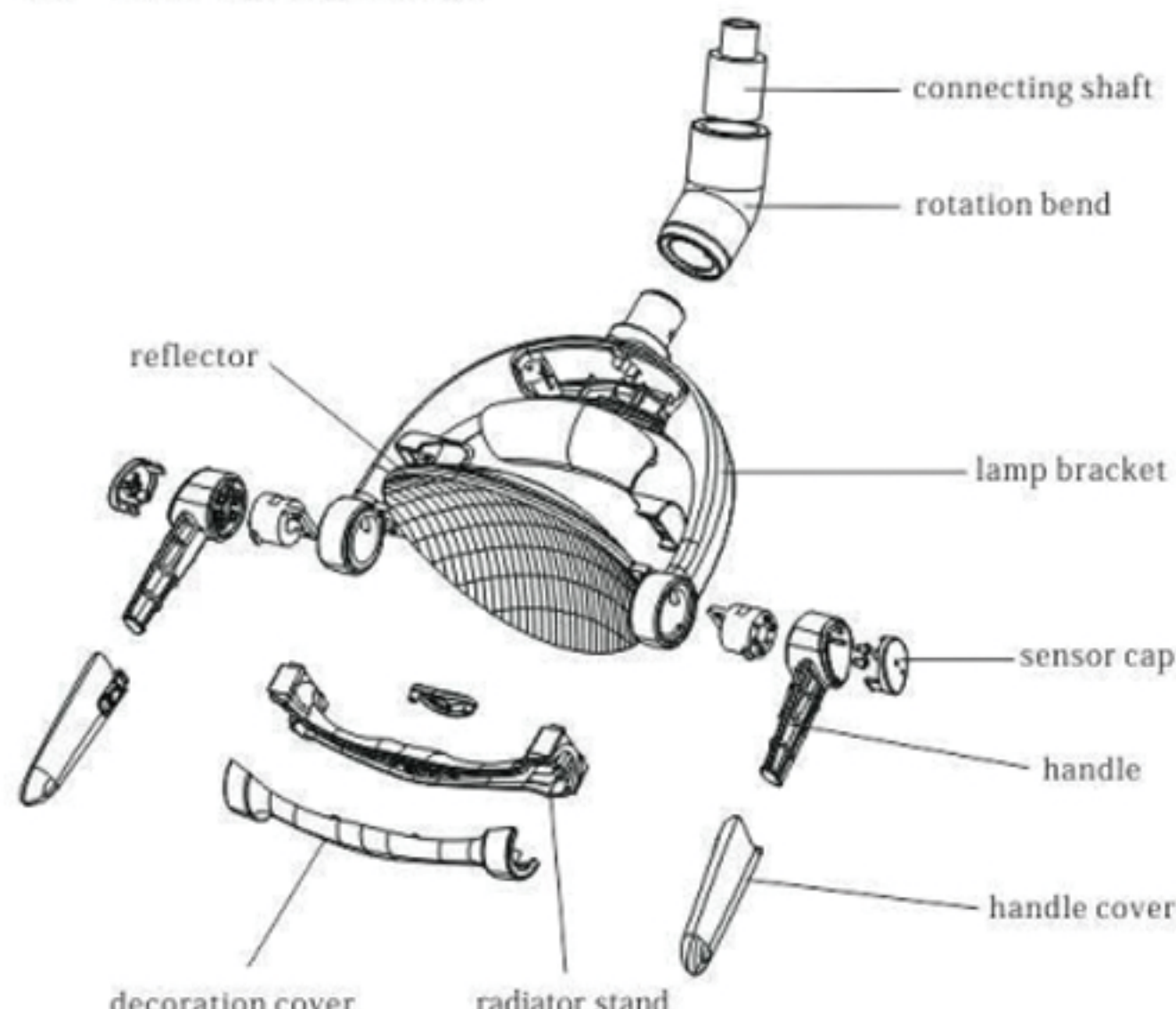

II. The technical specification

Voltage : AC12V-24V

Maximum Power : 10W

Highest intensity of illumination: $\geq 40000\text{Lux}$


Lowest intensity of illumination : $< 8000\text{Lux}$

Color temperature : 5000~6000K

Net weight : 980g



Gross weight : 1480g

III. The installation:

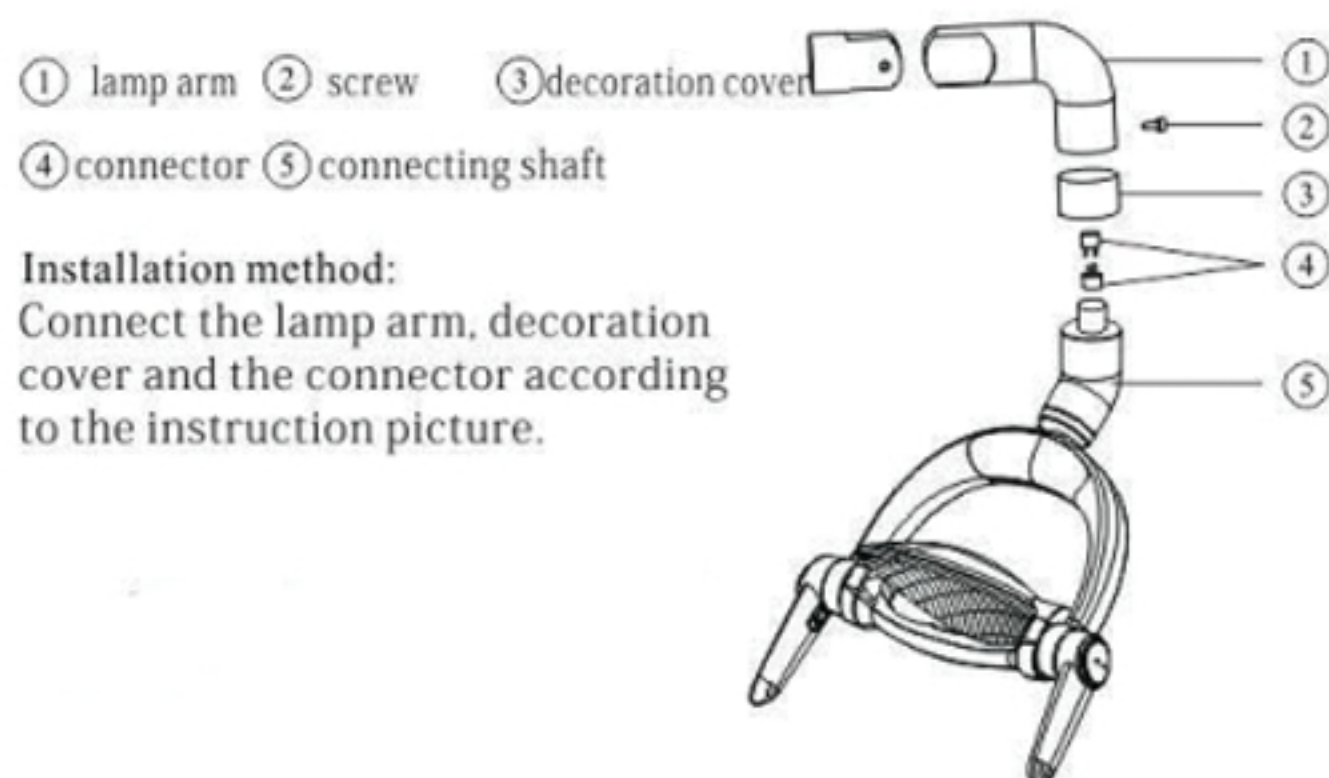


Installation method:
Connect the lamp arm, decoration cover and the connector according to the instruction picture.

IV. The manual:

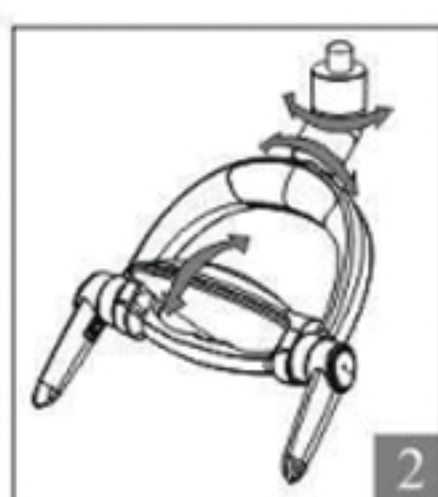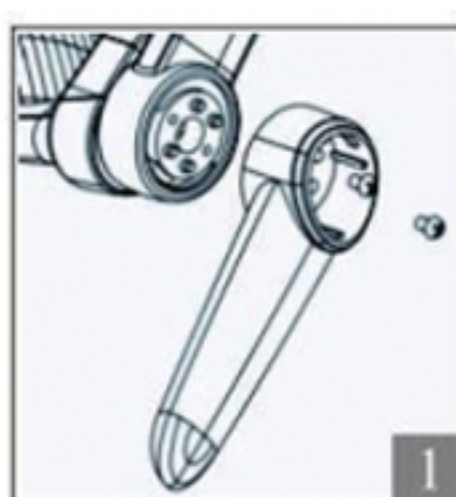


- 1, Adjust the angle of the handle to the accustomed position according to the instruction of the picture 1 . Loosen the screw to adjust the angle and tighten after the adjustment.
- 2, Hold the handle to adjust the light angle and distance to the best. (picture 2)

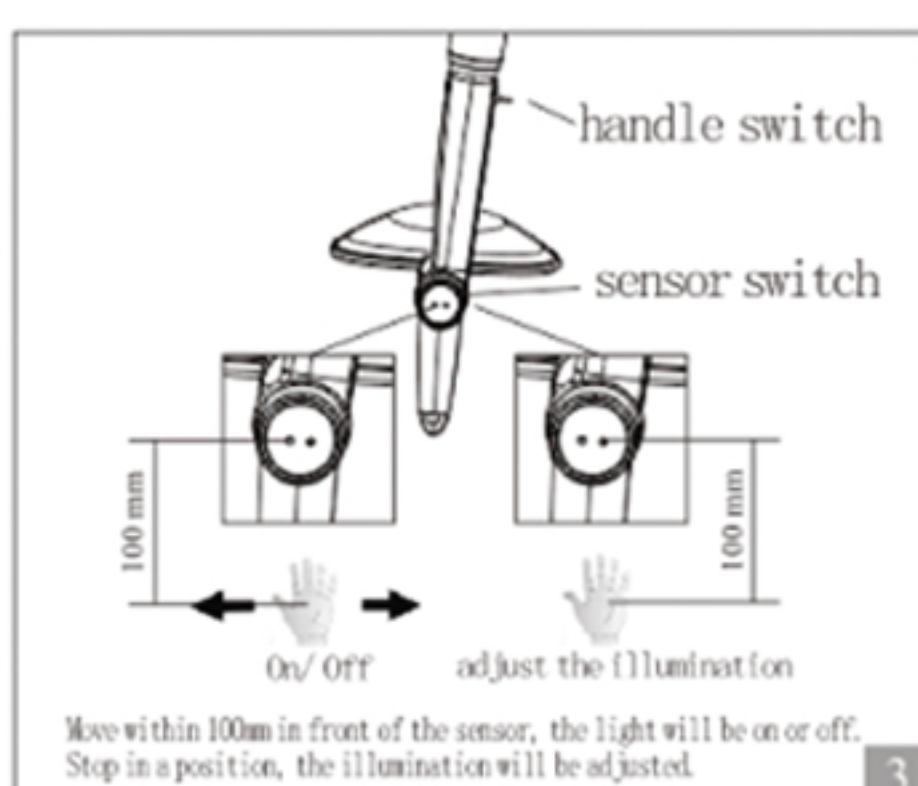

3. There are two kinds of switches for TR-966 . Touch the handle switch to turn on and off the lights. Hold the handle switch to change the intensity of the light to your target intensity and then the intensity will be saved in the LED light when turn on next time. Shake the hand in front of the sensor part to turn on or off. The strength of the light will be adjusted when hands stop.

When the intensity reaches to the maximum or minimum, there is a sound "Di" to remind and keep at this intensity for 3 seconds.

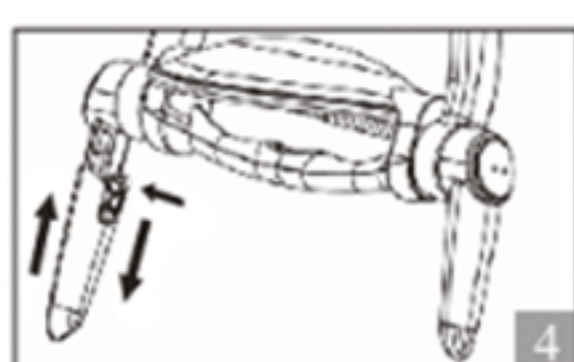

- 4, Press the button on the handle cover downward to take the cover off for cleaning and disinfection. After finished, push the handle in.

V. Maintenance & Attention

Clean via the 3-function sprayer or soft cotton and wet issue. The handle cover can be removed and disinfected under high temperature.

Notice:
The input voltage should be **AC12V-24V**. Wrong connection will cause the damage.
Don't use the caustic cleaning solution or the rough cloth to clean or rub the light.