


Manual of The Dental Operating LED Light TR-967

1, The structure of the light:

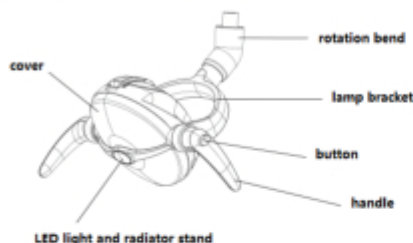

2, Installation:



- ① lamp arm ② screw ③ decoration cover ④ connector
⑤ connecting shaft

Installation method:

Connect the lamp arm, decoration cover and the connector according to the instruction picture.

Make sure the power source is AC 12V

3, The technical Specification:


Voltage: AC12V-24V Maximum Power: 10W Highest Intensity of Illumination $\geq 40000\text{Lux}$
Lowest intensity of illumination $< 8000\text{Lux}$ Color Temperature: 5300-5700K Net Weight: 1240g
Gross Weight: 1740g

4, The Manual:




1, Adjust the angle of the handle according to the picture 1 to any operation position u prefer. Loosen the screw to adjust the angle. After the adjustment, tighten the screw for normal use.

Adjust the light operation angle and distance by pulling the handles.

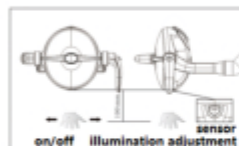

2, Move within 100mm in front of the sensor, the light will be on or off. Stop in a position within 100mm, the illumination will be adjusted.

When the intensity reaches to the maximum or minimum, there is a sound "Di" to remind and keep at this intensity for 3 seconds.

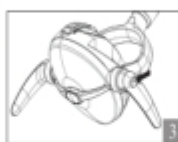

3, Press the button to take the handle off and make it to be disinfected. After the disinfection, push the handle back.

Maintenance & Attention:

Clean via the 3-function sprayer or soft cotton and wet tissue. The handle can be taken off and disinfected under the high temperature.

Notice:

The input voltage should be AC12V. Wrong connection will cause damage to the light. Don't use the caustic cleaning solution or the rough cloth to clean or rub the light.