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User's manual book

感谢您购买我们的产品，在使用之前请仔细阅读本产品说明书。
Thanks for choose our product, please read this manual book before use this unit.

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Specification & Technical

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Galilean Surgical Loupes is very popular applied to dental, plastics, neurological, ophthalmology, Otorhinolaryngology, also can be used for electronic and industry .

Our dental loupes is produced by high quality optical glass material with muli-coating.it can supply large field of views and deep depth of view.it is used for reduce dentist's eye, neck and shoulder fatigue, which is caused by long time lowerhead work .

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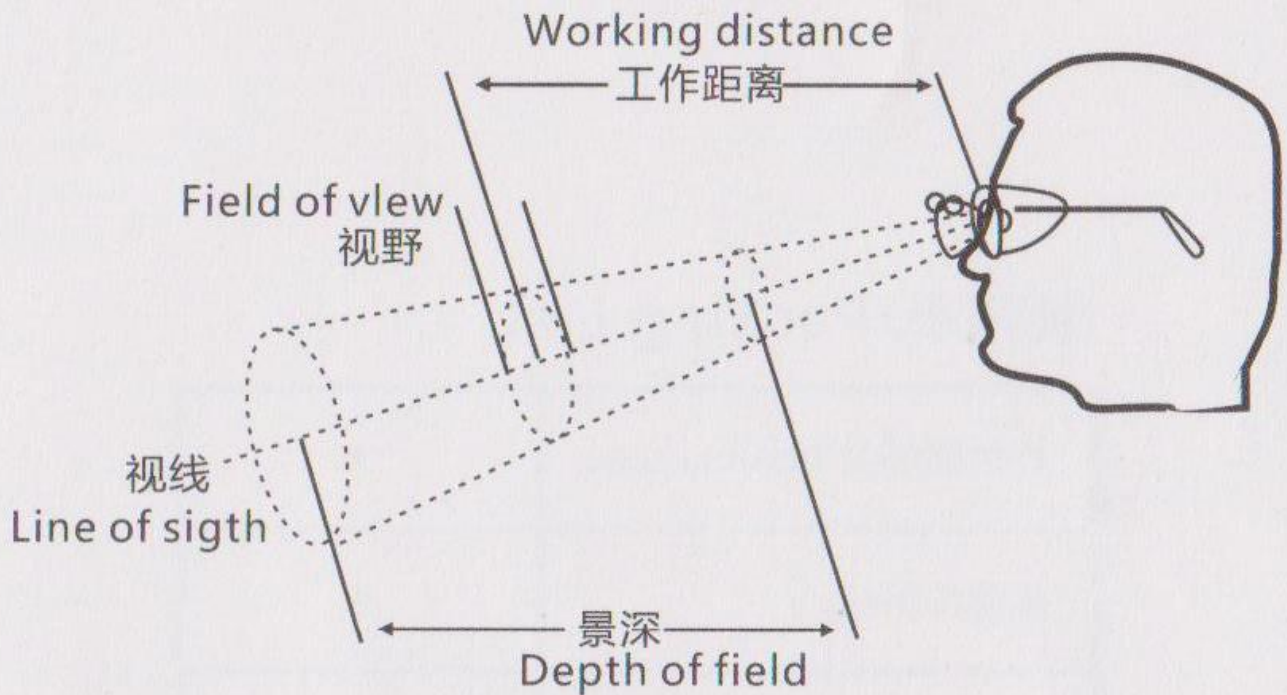
How to adjust and wear your loupes?

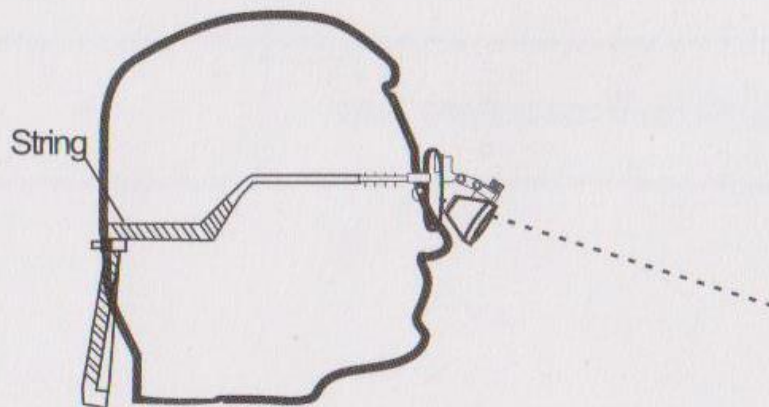
Illustration1

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Wear the loupes, fasten the string to wear comfortably and securely.

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Angled Hinge Adjustment

Adjust the hinge to see the object.

Note: When the lens more close to the frame, the image is more large and clearly.

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Interpupillary Adjustment

First adjust the interpupillary to maximum, then point at the object within the working distance. Rotate the middle nut slowly, when the circle you see become one from two, the object can be see clearly without ghost. Now it's the right interpupillary for you.

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Working distance

The loupes have a preset working distance, Be sure to position the object back and forth to find the right working distance for your focusing.

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The two side screws is for fixing the lens to get good image, please don't adjust.

装箱单/Packing List:

放大镜/Galillean Loupe	•
头绳/String	•
螺丝刀/Screwdriver	•
清洁布/Cleaning cloth	•
护镜盖/Protective cap	•